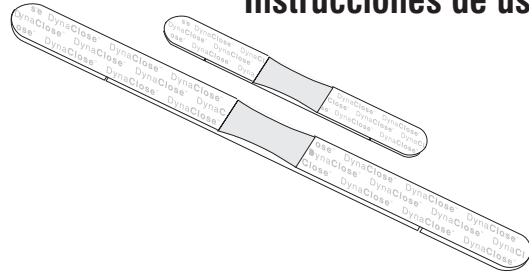
DynaClose®

Dynamic Skin Closure

Instructions for Use Directives d'utilisation Instrucciones de uso

Questions?

CANICA® 1-800-705-8312
or +1-613-256-0350
www.canica.com

Order Information

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STERILE R

Description	Part Number
DynaClose® 1/2" x 4 1/2" (13 mm x 114 mm) 4 pcs/pouch, 15 pouches/box For wounds up to 1" (2.5 cm) in width	DWC24/Box
DynaClose® 3/4" x 8 1/2" (19 mm x 216 mm) 4 pcs/pouch, 15 pouches/box For wounds up to 2" (5 cm) in width	DWC38/Box

CEpartner4U
3951DB; 13. NL.
Tel: +31 (0)6.516.536.26

EC REP

Canica Design Inc.
Head Office
36 Mill Street, Almonte, ON Canada K0A 1A0
U.S. Distribution Center
814 Proctor Ave, Ogdensburg, NY USA 13669-2205

Phone: 1-613-256-0350
Fax: 1-613-256-0360
Email: sales@canica.com
Web: www.canica.com
Toll Free: 1-800-705-8312

Questions?

1-800-705-8312

or call +1-613-256-0350

TRI-1D1368RD

Last revised March 19, 2010

About DynaClose®

English

Introduction

DynaClose is available in a variety of sizes to close defects up to 5 cm in width.

Indications

DynaClose is indicated for use in preventing, controlling, reducing, and closing skin defects. No known contra-indications.

Warnings

1. If trimming ends of adhesive fabric, reduce tension to prevent skin damage.
2. If allergic reaction occurs, discontinue use.
3. If blistering occurs, relocate at reduced tension or discontinue use.

NO REUSE: DynaClose is not reusable. The adhesive tape uses a single-contact adhesive that is not designed to withstand repeated use.

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Cautions for Extremity Use

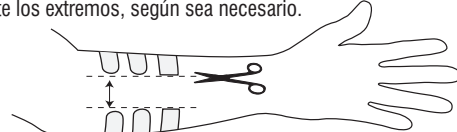
Do not allow DynaClose to encircle an extremity as this will restrict circulation. Choose a smaller DynaClose or trim ends as required.

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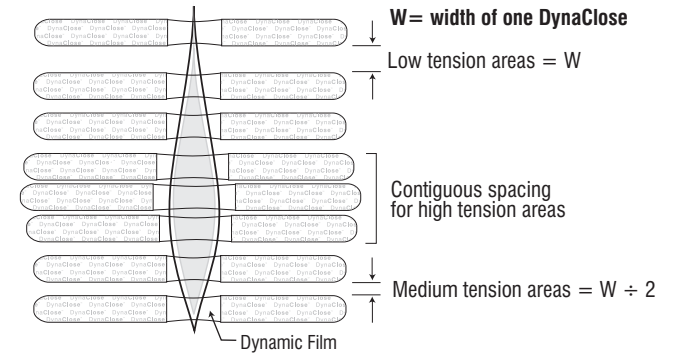
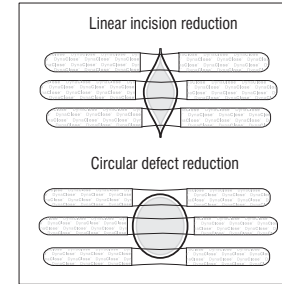
English

Wound Preparation

Best results are achieved when margins are mobile and free of granulation tissue.

Skin Preparation

Shave and clean the skin area extending 12 cm from the wound edge. Wipe skin with rubbing alcohol to improve adhesion.

Dressings

Deep wounds require dressings placed under the dynamic film. Shallow and approximated wounds require dressings placed on top of the dynamic film.

Note: Refrain from using small dressings that may get left behind under rapidly closing wound edges. Avoid any greasy, petroleum jelly impregnated dressings which will reduce adhesive effectiveness.

Closure

When wound edges touch, continue applying DynaClose until wound is healed and skin integrity is restored (up to 29 days). To prevent dehiscence and reinforce closed incisions, use SutureSafe (see www.canica.com for more information).

Application

- 1** Maximum Tension Limiter

Release Liner (center section)

Peel Center
Peel and remove center section of bottom release liner.
- 2** Stretch

Maximum tension (flat)

Green Indicator Line

Stretch
Grip ends and pull DynaClose until the desired tension is achieved, ensuring not to exceed maximum stretch allowed by the limiter. Center wound between green lines.
For low tension wounds and for fragile or ischemic skin, individual DynaClose tensions can be reduced by not pulling maximum tension limiter fully taut during installation.
- 3** Position and Press

Red Indicator Line



Position and Press
With the DynaClose still tensioned, press down between the green and red lines to apply to skin.
- 4** Peel Ends and Apply

Peel Ends and Apply
Peel off outer release liners and complete application to skin. For optimal adhesion, gently rub the adhesive area.
- 5** Remove Maximum Tension Limiter

Remove Maximum Tension Limiter
Peel limiter from the ends near the red lines, and discard. Repeat steps 1-5 along entire length of wound.
- 6** Replacing DynaClose

Replacing DynaClose
Replace DynaClose one at a time as required to maintain closure effectiveness and avoid loss of progress. Replace when the dynamic film length approaches half of its applied length. Peel ends back toward the wound for painless removal.
Note: Some adhesive residue may remain on skin if DynaClose is removed when wet. Remove remaining adhesive residue with a skin adhesive remover or let dry then rub off.

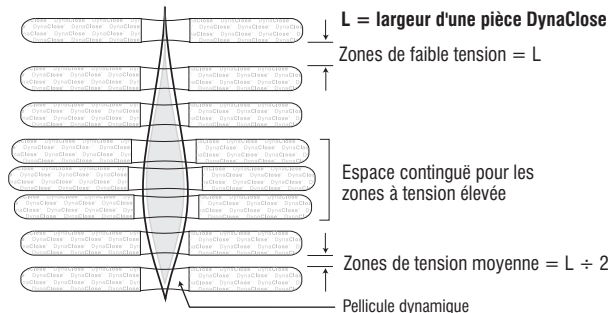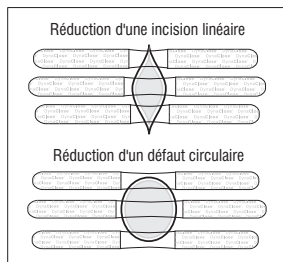DWC38 - Apply at: 5 cm	Replace at: 3 cm
DWC24 - Apply at: 2.5 cm	Replace at: 1.5 cm

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