

KIKUHIME

Sub Bandage Pressure Monitor



Kikuhime is a portable sub-bandage pressure monitoring device. A sensor connected to the monitor is positioned beneath a bandage for an accurate (+/- 1mmHg) determination of pressure exerted on the sensor.

The Kikuhime unit can also be used for other pressure related checks e.g. determining sacrum pressure when checking effectiveness of your pressure ulcer prevention regime. (The large sensor is recommended.)

Rent or Buy!

How to use?



Simply place the pressure sensor in the Easy Glide sleeve and locate sensors at the places you wish to measure pressure. Wrap the bandage or compression therapy around the limb.



Wrap the compression garment around the limb.

Check the pressure(s) if the multi-sensor is used.



Pull the smooth slip sleeve containing the sensors out from under the compression garment.

Applications?

Training and Education: Workshops on bandaging techniques. Ideal for testing skills of participants at the end as well as during the workshop.

Knowledge: Variable standards can cause confusion (e.g. a compression stocking labelled 19mmHg may actually provide more compression than another brand labelled 28mmHg). Get an accurate compression reading and know what your patients are getting.

Research: Because Kikuhime provides simple, accurate reading, the investigator can focus on research questions rather than complex procedures.

Routine Monitoring: Perhaps achieving optimal compression is critical - maybe for a difficult patient. Or maybe you need to routinely evaluate sacral pressures.

Features?

- Very simple to use
- Accurate pressure reading
- Portable size (9 x 6 x 2 cm, weight 90g)
- Hold function (for steady readout)
- Long 5yrs battery life
- Clip for secure attachment

Rental

Short Term:

3 day package or daily rate
Ideal for single training session or workshops.
Delivery by Express Mail.
Return it via Express Mail the same way!

Long Term:

Month or weekly rate.
Ideal for large projects or audits.

Product Codes

Product Description	Code
Kikuhime Kit - Kikuhime Unit - 1 Small Sensor - 1 Large Sensor - Calibration tools (screwdriver and syringe) - Carry case - Easy Glide (for removing a sensor from beneath a bandage)	KK-Base
Accessory: 3-way valve*	KK-3Way
Accessory: Small sensor	KK-Sml
Accessory: Large sensor	KK-Lrg

* Up to 3 sensors can be attached to one Kikuhime at a time by using the optional 3-way valve accessory.

mediGroup Australia Pty Ltd
Level 1, 530 Lt Collins St
Melbourne VIC 3000
P: 1300 362 534 F: 1300 362 482
sales@medigroup.com.au
www.medigroup.com.au

Your Local Distributor:

