

LUCKi

Leg Ulcer Compression Kit

HEALTH SUPPORT MULTI-LAYER COMPRESSION SYSTEM
BY CAROLON



Make 2 decisions.

1. Select the required **Compression** level
30-35 mmHg (Class II) or
35-40 mmHg (Class III) at the ankle.
2. Select the **Style** for the outer stocking
Option 1: Black sock
Option 2: Beige knee high hosiery

CARE AND USAGE INSTRUCTIONS

To maintain optimum condition and elasticity:

For best results wash inner and outer daily.

HOT WASH (> 60°C)

or

WARM WASH (40°C) and tumble dry on medium.

Please wash stockings in laundry bags to protect them.

Do not wash in cold water.


Wear inner overnight when at rest/sleep.

Wear inner and outer during the day when ambulatory.

Take 3 measurements.

1. Measure the smallest ankle circumference
2. Measure the largest calf circumference. 1 & 2 give you your **size**.
3. Measure the distance from the bend in the knee to bottom heel to get your **length**.

Ankle (cm)	Calf (cm)	Size	Calf height (knee to heel)		Length
			Beige hosiery	Black sock	
18 - 20	25.5 - 33	A	To 38 cm	To 40.5 cm	S
			Over 38 cm	Over 40.5 cm	R
20 - 22.5	30.5 - 38	B	To 40.5 cm	To 43 cm	S
			Over 40.5 cm	Over 43 cm	R
22.5 - 25.5	35.5 - 43	C	To 43 cm	To 45.5 cm	S
			Over 43 cm	Over 45.5 cm	R
25.5 - 28	40.5 - 48.5	D	To 45.5 cm		S
			Over 45.5 cm		R
28 - 30.5	45.5 - 53	E	To 45.5 cm		S
			Over 45.5 cm		R
30.5 - 33	50 - 58.5	F	To 45.5 cm		S
			Over 45.5 cm		R
33 - 35.5	55.5 - 66	G	To 45.5 cm		S
			Over 45.5 cm		R

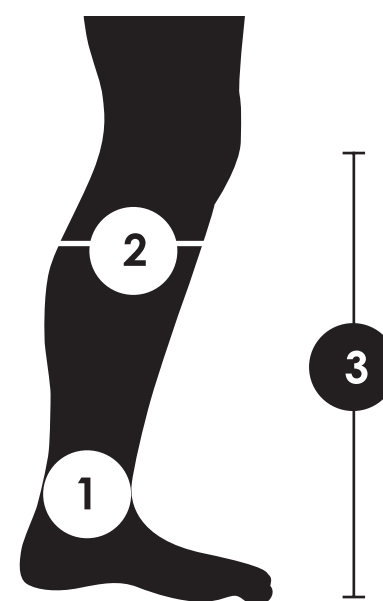

Now use the size, length, compression class and style to order.

E.g. HSML 30-35 A S Beige

Note: Packaging contains two inners and two outer stockings.

mediGroup Australia Pty Ltd

P: 1300 362 534 F: 1300 362 482
sales@medigroup.com.au

Your local distributor: