

# CAP/ATS

## Anti-Embolism Stocking



Put your hand inside the foot area of the stocking.



Turn stocking inside out, stopping at heel. Spreading fingers, leave foot area of stocking over the palm of your hand.



Stretching heel area sideways, fit stocking over patient's foot and heel. Be sure patient's heel is in centre of stocking's blue heel pocket.



Grasp top of stocking and pull over foot and ankle. Work stocking up over leg. Smooth and wrinkles. Make sure knit change at ankle is located at the ankle rotation.



For thigh stocking turn gusset over femoral. Insert finger in foot inspection hole and pull up, properly positioning fabric over foot, relieving possible pressure on toes.



SIEGEL profile compression 18 mmHG at the ankle.

### KNEE LENGTH

1. Measure narrowest ankle circumference. Use the ankle to determine the proper size. Do not apply if calf size is two sizes larger than ankle size.

2. Measure largest calf circumference.

2A. Measure distance from bend in knee to bottom of heel for length.

Refer to chart for size.

### THIGH LENGTH

1. Measure narrowest ankle circumference. Use the ankle to determine the proper size. Do not apply if calf size is two sizes larger than ankle size.

2. Measure largest calf circumference.

3. Measure largest thigh circumference. If thigh circumference exceeds 30", refer to ATS Style.

3A. Measure distance from gluteal crease to bottom of heel for length.

Refer to chart for size.

## CAP Knee Length

Sizes highlighted will fit most of your patients

Product Description	Ankle (cm)	Calf (cm)	Thigh (cm)	Length (cm)	Product Code	Size Colour Code (TOP)	Size Colour Code (TOE)
X-Small	15 - 17.5	23 - 30	N/A	31 - 36.5	CAP 501	Natural	N/A
Small Short				30.5 - 36	CAP 510	Yellow	Pink
Small Regular	17.5 - 20	25.5 - 33	N/A	36 - 40.5	CAP 511	Yellow	Natural
Small Long				40.5 - 46	CAP 512	Yellow	Orange
Medium Short				33 - 38	CAP 520	Natural	Pink
Medium Regular	20 - 22.5	30.5 - 38	N/A	38 - 43	CAP 521	Natural	Natural
Medium Long				43 - 48.5	CAP 522	Natural	Orange
Large Short				35.5 - 41	CAP 530	Navy	Pink
Large Regular	22.5 - 25.5	35.5 - 43	N/A	41 - 45.5	CAP 531	Navy	Natural
Large Long				45.5 - 51	CAP 532	Navy	Orange
X-Large Short				35.5 - 41	CAP 540	Green	Pink
X-Large Regular	25.5 - 28	40.5 - 48	N/A	41 - 45.5	CAP 541	Green	Natural
X-Large Long				45.5 - 51	CAP 542	Green	Orange
XX-Large Short				35.5 - 41	CAP 550	Purple	Pink
XX-Large Regular	28 - 30.5	46 - 53	N/A	41 - 45.5	CAP 551	Purple	Natural
XX-Large Long				45.5 - 51	CAP 552	Purple	Orange
XXX-Large Short	30.5 - 35.5	51 - 71	N/A	35.5 - 41	CAP 560	Natural	N/A
XXX-Large Medium	30.5 - 35.5	51 - 71		41 - 45.5	CAP 561		
XXX-Large Long	30.5 - 35.5	51 - 71		45.5 - 51	CAP 562		

## CAP Thigh Length

Product Description	Ankle (cm)	Calf (cm)	Thigh (cm)	Length (cm)	Product Code	Size Colour Code (TOP)	Size Colour Code (TOE)
X-Small	15 - 17.5	23 - 30	33 - 48	66 - 73.5	CAP 601	Natural	N/A
Small Short				66 - 74	CAP 610	Yellow	Pink
Small Regular	17.5 - 20	25.5 - 33	40.5 - 51	74 - 79	CAP 611	Yellow	Natural
Small Long				79 - 86	CAP 612	Yellow	Orange
Medium Short				66 - 74	CAP 620	Natural	Pink
Medium Regular	20 - 22.5	35.5 - 38	48 - 61	74 - 79	CAP 621	Natural	Natural
Medium Long				79 - 86	CAP 622	Natural	Orange
Large Short				66 - 74	CAP 630	Navy	Pink
Large Regular	22.5 - 25.5	35.5 - 43	58.5 - 68.5	74 - 79	CAP 631	Navy	Natural
Large Long				79 - 86	CAP 632	Navy	Orange
X-Large Short				66 - 74	CAP 640	Green	Pink
X-Large Regular	25.5 - 28	40.5 - 48	63.5 - 76	74 - 79	CAP 641	Green	Natural
X-Large Long				79 - 86	CAP 642	Green	Orange

## ATS Thigh Length

FOR HIP SURGERY, EXTRA LARGE THIGHS OR ABNORMALLY SHAPED LEGS

Product Description	Ankle (cm)	Calf (cm)	Thigh (cm)	Length (cm)	Product Code	Size Colour Code (TOP)	Size Colour Code (TOE)
Small Regular	17.5-19.5	28.32.5	44.5-63.5	69-76.5	ATS 311	Yellow	Natural
Small Long				76.5-84	ATS 312	Yellow	Orange
Medium Regular	19.5-21.5	32.5-35.5	53.5-71	69-76.5	ATS 321	Natural	Natural
Medium Long				76.5-84	ATS 322	Natural	Orange
Large Regular	21.5-23.5	35.5-39.5	61-80	69-76.5	ATS 331	Navy	Natural
Large Long				76.5-84	ATS 332	Navy	Orange
X-Large Regular	23.5-25.5	39.5-42.5	70-84	69-76.5	ATS 341	Green	Natural
X-Large Long				76.5-84	ATS 342	Green	Orange
XX-Large Reg	25.5-30	42.5-50	76-91.5	69-76.5	ATS 351	Purple	Natural
XX-Large Long				76.5-84	ATS 352	Purple	Orange
XXX-Large Reg	30-35	51-60	91.5-106	66-74	ATS 360	Natural	N/A