



## Foot WAFFLE® Brand Air Cushion

*Comfort, prevention and healing therapy thru Stage IV pressure ulcers. Deep tissue injury protection.*

### Indications for use:



1. Measure widest part of calf (in inches) with dressing or antithrombetic products on, if applicable.



2. Place leg in product keeping heel and foot perpendicular to the product.



3. After leg is placed in the product, secure Velcro® straps.



4. Strap closest to the foot is covered in a soft, cotton material for comfort and protection.



5. Check that the patient's heel is off the bed. Add air to the heel flap, if necessary.



6. Insert hand along the side of the foot, down toward the malleolus. Fingers should slide into opening without difficulty.



7. To protect patients who 'dig' their heels, release the heel cushion, bring up the strap to the end of the heel flap and reseal.

Product Description	Calf Size (cm)	No. in Box	Product Code
Foot Waffle® SMALL	25.5 - 29.2	5	303FW
Foot Waffle® MEDIUM	30.5 - 35.6	5	304FW
Foot Waffle® LARGE	36.8 - 45.7	5	305FW
Waffle® M.A.D Pump	N/A	1	540MAD
Waffle® Mini Pump	N/A	1	530HP


### Patient Care

Daily hygiene consists of removing the product and washing, drying and inspecting the patient's foot and leg.

To achieve a complete continuum of care, the Foot WAFFLE® should accompany the patients to and from hospitals, nursing homes and home care.

### Product Care

The product can be wiped off and cleaned with soap/water or the germicidal used at your facility. Do not use cleaning solvents containing alcohol or petroleum distillates.

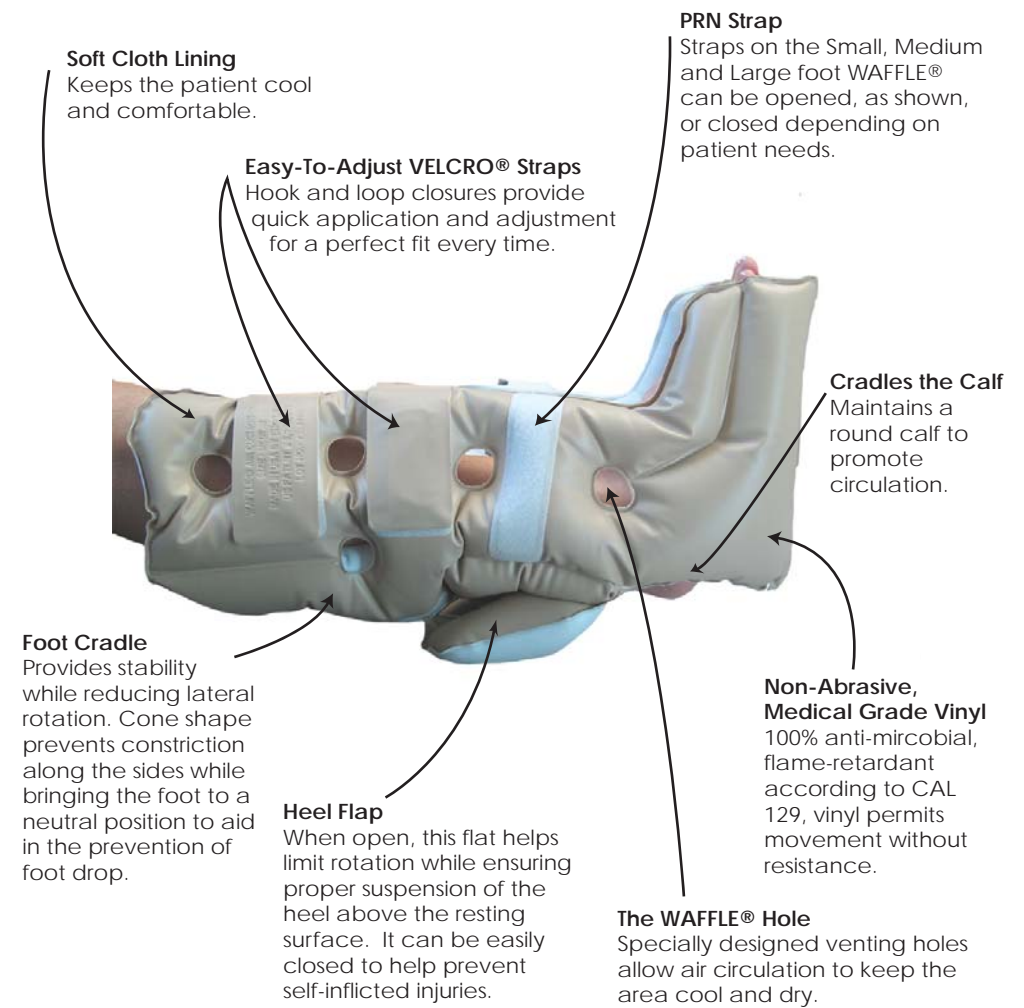

Manufactured by:

